**MODE D'EMPLOI - INSTRUCTIONS FOR USE
ISTRUZIONI D'USO - MANUAL DE INSTRUCCIONES**

Atomiseur – Atomizer – Atomizzatore - Atomizador

ASEPT  JET S


Rév. 120121

ASEPTOJET S

A.	DESCRIPTIF DE L'APPAREIL – EQUIPMENT DESCRIPTION – DESCRIZIONE DELL'APPARECCHIATURA - DESCRIPCION DEL EQUIPO.....	3
B.	SCHEMA ELECTRIQUE - WIRING DIAGRAM – SCHEMA ELETTRICO ESQUEMA ELÉCTRICO.....	4
C.	MODE D'EMPLOI.....	6
D.	INSTRUCTIONS FOR USE	12
E.	ISTRUZIONI D'USO	18
F.	MANUAL DE INSTRUCCIONES	24


Visuels non contractuels / Non-contractual images
Foto non contrattuali / Imágenes no contractuales

A. DESCRIPTIF DE L'APPAREIL – EQUIPMENT DESCRIPTION – DESCRIZIONE DELL'APPARECCHIATURA - DESCRIPCION DEL EQUIPO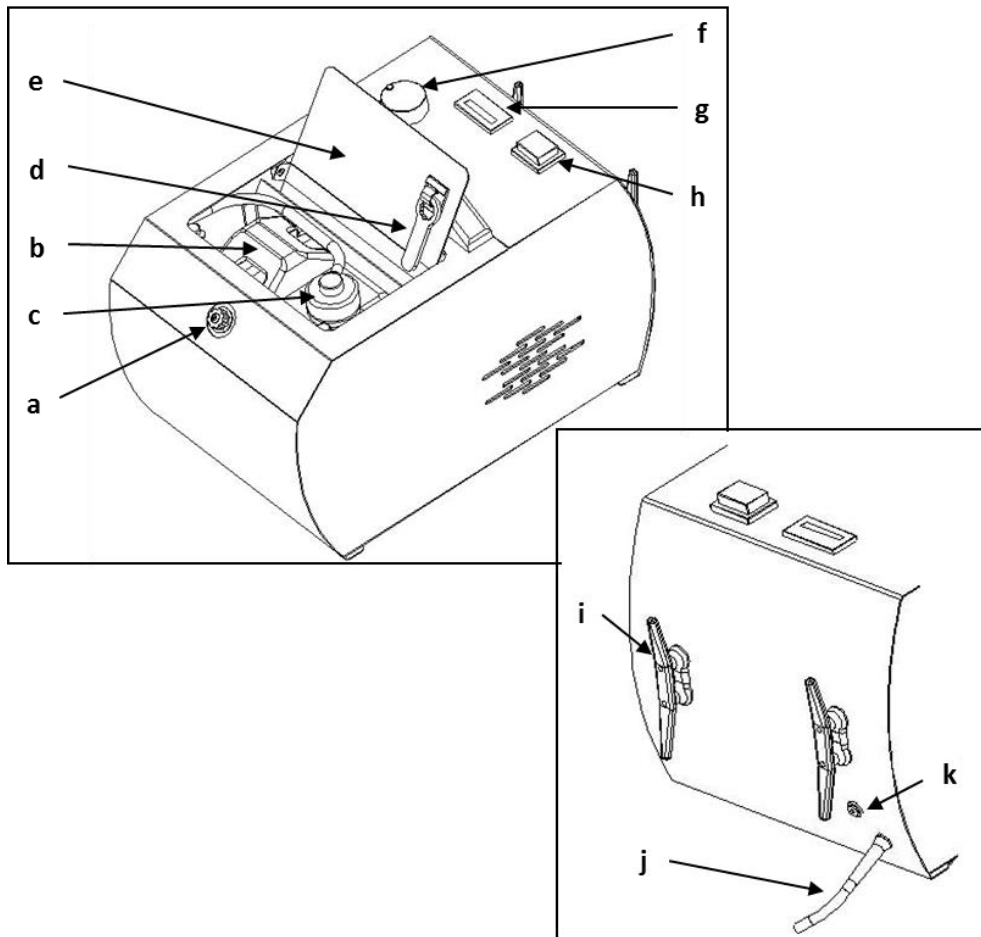

<i>N°</i>	<i>DESCRIPTION</i>	<i>DESCRIPTION</i>	<i>DESCRIZIONE</i>	<i>DESCRIPCIÓN</i>
a	Buse	Nozzle	Ugello	Boquilla
b	Réservoir - 2 litres	Tank - 2 litres	Serbatoio - 2 litri	Depósito – 2 litros
c	Bouchon	Cap	Tappo	Capuchón
d	Clé de buse	Key for nozzle	Chiave ugello	Llave de la boquilla
e	Trappe pivotante	Pivoting cover	Sportello	Trampilla
f	Sélecteur de volume	Volume knob selector	Selettore di volume	Selector de volumen
g	Compteur horaire	Hour meter	Conta ore	Contador de horas
h	Interrupteur marche/arrêt	Switch ON/OFF	Interruttore ON/OFF	Interruptor encendido/apagado
i	Enrouleur	Winding system	Avvolgitore	Rebobinador
j	Cordon d'alimentation	Electric cord	Cavo di alimentazione	Cable de alimentación
k	Disjoncteur thermique 10A	Thermal circuit breaker 10A	Disgiuntore termico 10A	Disyuntor térmico 10A

B. SCHEMA ELECTRIQUE - WIRING DIAGRAM – SCHEMA ELETTRICO ESQUEMA ELÉCTRICO

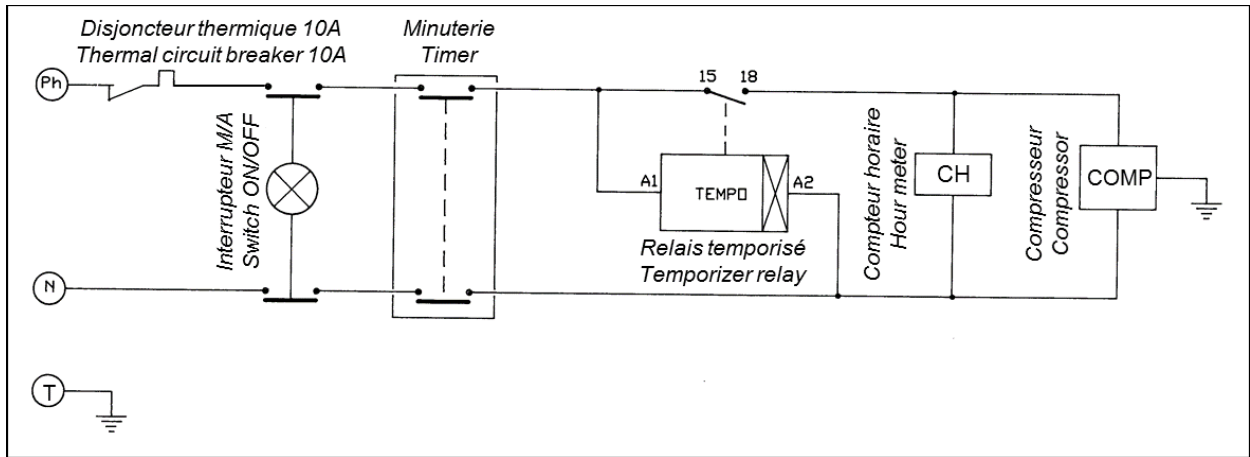ASEPTOJET S

A.	DESCRIPTIF DE L'APPAREIL – EQUIPMENT DESCRIPTION – DESCRIZIONE DELL'APPARECCHIATURA - DESCRIPCION DEL EQUIPO.....	3
B.	SCHEMA ELECTRIQUE - WIRING DIAGRAM – SCHEMA ELETTRICO ESQUEMA ELÉCTRICO.....	4
C.	MODE D'EMPLOI.....	6
1.	CONSIGNES DE SECURITE	6
1.1.	UTILISATION	6
1.2.	CONSIGNES DE SECURITE	6
1.3.	ELIMINATION DES DECHETS	7
2.	DECLARATION DE CONFORMITE CE	7
3.	CARACTERISTIQUES TECHNIQUES.....	8
4.	INSTALLATION ET MISE EN SERVICE	8
4.1.	DEBALLAGE DU PRODUIT.....	8
4.2.	INSTALLATION ET UTILISATION.....	9
4.3.	TRAITEMENT CURATIF	10
4.4.	TRAITEMENT PREVENTIF	10
5.	MAINTENANCE	10

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1. CONSIGNES DE SECURITE

1.1. UTILISATION

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Translation into English from original in French.

D. INSTRUCTIONS FOR USE

1. SAFETY INSTRUCTIONS

1.1. USE

ASEPTOJET S is intended for the disinfection of surfaces and volumes by air, through the atomization of a disinfectant liquid.

This equipment has to be operated only by professionals.

Note: disinfectant properties according to the liquid used - consult the product data sheet.

ASEPTOJET S has to use in the conditions and for the uses described in this manual. Any other use that will be made of it is considered inappropriate. In this case, the manufacturer declines all liability and immediately cancels the equipment warranty.

It is imperative to **read carefully the instructions for use** before installation and use. Retain the manual for future reference.

1.2. SAFETY INSTRUCTIONS

Pictograms' meaning used in these instructions for use:

	General warning		Electrical hazard
--	-----------------	--	-------------------

Observe following safety instructions:

	<ul style="list-style-type: none"> - The unit has to be connected to an electrical installation in accordance with the applicable standard in the country in which it is being used. This installation has to provide protection against overcurrent, overload and ground faults. - Connect it directly to socket. Do not use multiple sockets. - Ensure that the electric cord is in perfect condition to prevent short circuits. - Before any technical intervention, the unit has to be switched off and disconnected from the mains power supply. - Do not spill liquid on the ventilation openings or inside the equipment.
	<ul style="list-style-type: none"> - Not exceed the maximum tank's volume (2 litres). - Do not operate equipment without liquid in the tank. This may result in malfunction of the unit and damage to certain components. - Mixing of disinfectants is not recommended - dangerous mixtures may form. Observe the manufacturer disinfecting's instructions.
	<ul style="list-style-type: none"> - Use only genuine spare parts. The use of non-original spare parts voids any warranty for your device. - Do not introduce stranger objects inside the unit during the maintenance operations. It is forbidden to modify the material without authorization. - Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. - For whichever information concerning installation, maintenance and use, contact the customer's service UGIN DENTAIRE.

1.3. WASTE DISPOSAL

Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.

The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.

2. EC DECLARATION OF CONFORMITY

The equipment's EC declaration of conformity is provided as appendix of instructions for use.

Content of the declaration EC:

- Name and address of the manufacturer:
UGIN DENTAIRE
25 rue de la Tuilerie
38170 SEYSSINET-PARISSET – FRANCE
- UGIN DENTAIRE declares, on its own exclusive responsibility, that the product:
 - Atomizer **ASEPTOJET S**
 - Serial number: *see the equipment's EC declaration of conformity*
- Fulfills the essential requirements of health and safety of the following directives:
 - Machinery directive 2006/42/EC
 - Electromagnetic compatibility directive 2014/30/EU

3. TECHNICAL INFORMATION

ASEPTOJET S is intended for the disinfection of surfaces and volumes by air, through the atomization of a disinfectant liquid. This equipment has to be operated only by professionals and in the conditions described in this manual.

Note: disinfectant properties according to the liquid used - consult the product data sheet.

The atomizer **ASEPTOJET S** is completely automatic and easy to use. Its 60 seconds temporizer allows the user to leave the room before the atomizer functioning.

TECHNICAL DATA	
Dimensions	
- Width	245 mm
- Height	280 mm
- Depth	400 mm
Weight (empty)	12 Kg
Power consumption	300 W
Supply voltage	230 V
Frequency	50 / 60 Hz
Volume of tank	2 litres
Maximum volume to be treated	450 m ³
Consumption of liquid	1800 ml / hour
Autonomy (2 g/m³)	1000 m ³
Dry compressor (without oil)	1.6 m ³ /hour
Noise level	< 70 db(A)
Timer	0 to 30 minutes
Temporizer	60 seconds

4. INSTALLATION AND START-UP

4.1. UNPACK

- Unpack **ASEPTOJET S** and check that it is in perfect order.
- Any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note.

NOTE: keep the original packaging to transport.

4.2. INSTALLATION AND USE

Note: for marks, see chapter A – equipment description.

1. **Open the pivoting cover (e).**
2. **Unscrew the tank's cap (c).**
3. **Remove the tank (b) of equipment and fill-in with the disinfectant liquid choose (use a suitable funnel if necessary). Attention to do not exceed the maximum tank's volume (2 litres).**

IMPORTANT: not exceed the maximum tank's volume (2 litres).

4. **Replace the tank** in the equipment (in the space provided).
5. **Screw the tank's cap (c) and close the pivoting cover (e).**
6. Make sure that the mains voltage complies with the voltage indicated on the rating plate, **connect the power cord (j)** to a 10/16 A grounded power socket.
7. Put the equipment in the **middle of the room to be treated**, and keeping a **distance of at least 1 meter** from anything furniture and wall.
8. Activate the **ON/OFF switch (h).**
9. Make an estimate of the room's volume in m³. Set the **volume selector (f)** according to the room's volume to be treated.

IMPORTANT: turn the volume selector clockwise direction. Never selector manually counter-clockwise direction: may cause the deterioration of the selector.

10. **The spraying starts 60 seconds** after the volume is selected, which allows the user to leave the room before start the product's spraying.
11. The spraying continues for the programmed time (according to selected volume), then **stops automatically at the end of the cycle.**

If the tank contains liquid from a previous use, make sure that it contains enough liquid to perform the operation. **If necessary, add the required amount of the same product** (do not exceed the maximum volume of 2L).

IMPORTANT: do not operate equipment without liquid in the tank.

This may result in malfunction of the unit and damage to certain components.

- **CAUTION: mixing of disinfectants is not recommended** - dangerous mixtures may form. Observe the manufacturer disinfecting's instructions. Consult the product data sheet and the material safety data sheet (MSDS).
- **Do not use flammable, corrosive, toxic liquids.**
- Before use, consult the disinfectant product's technical data sheet to **check the compatibility with the floor/wall covering and the furniture/materials of the room to be treated** (for example, certain liquids do not recommend using them on copper, copper alloys or light metals such as aluminium). If in doubt, it is recommended that you test a patch of the surface to be cleaned.
- **Room's temperature and humidity conditions can cause the aerosols of the disinfectant liquid to condense.** It is recommended to use Aseptojet S in a room at ambient temperature (15°C - 30°C).

4.3. CURATING TREATMENT

To provide a maximum decontamination of the room, it is strongly advised for the **first usage**, to proceed to a complete curative treatment. For this operation, we advise to set the volume selector (f) to **450m³ / 30 minutes** (for a room's volume limited to 100m³).

4.4. PREVENTIVE TREATMENT

Set the volume selector (f) **on the estimated room's volume to be treated (in m³)**.

5. MAINTENANCE

Before to carry out the maintenance work and/or cleaning: disconnect from the main power supply (switch OFF and unplug to mains socket).

Observe the following recommendations to ensure the longevity of the device:

- Do not use abrasive, solvents or flammables products for cleaning (outside and inside).
- Clean the machine outside with a soft, dry cloth.
- Make a **visual inspection of the nozzle's connection hoses with the tank (n° 26) and with the compressor (n° 29)**. If damaged: do not use the equipment before replacing the defective part(s).
- The **air dry compressor (without oil)** does not require any particular care.
- The **nozzle** is a very important part of this equipment. The length of the spray must be at least of 1m to 1,50m. In case of problems, the nozzle must be cleaned: nozzle's maintenance.

NOZZLE'S MAINTENANCE – steps

- The key for nozzle provided (d) allows the fast taking apart,
- Make a ¼ of a turn (counter clock wise) with the key for nozzle (n° 4),
- Take off the nozzle's insert (n° 3) and central part (n° 2),
- Let it soak nozzle's insert (n° 3) and central part (n° 2) during 2 hours in bleach, then rinse thoroughly with clean water,
- Proceed in reverse order for assembly.

If the failure persists, contact the after-sale service.
Nozzle's replacement may be necessary.

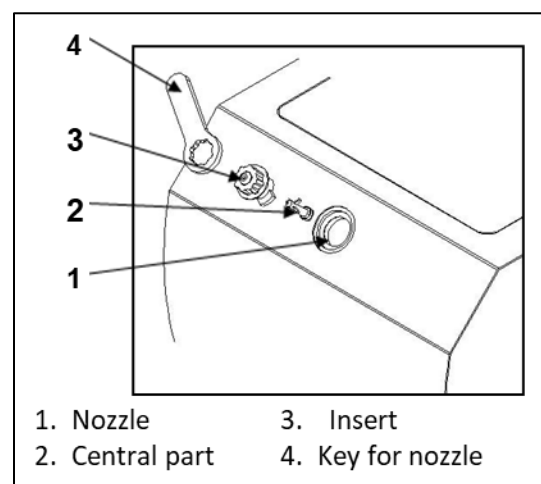
Figure 1: nozzle's maintenance

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