**MODE D'EMPLOI - INSTRUCTIONS FOR USE**

# **MINICLEAN 2**

**POLISSAGE ELECTROMAGNETIQUE  
*ELECTROMAGNETIC POLISHING***

CE

**UGiN<sup>®</sup>**  
dentaire

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*Translation into English from original in French*

**IMPORTANT!**

Before installing the apparatus **read carefully this manual** and keep it for future consultations.

**MINICLEAN 2** must only be used in the manner and for the purposes described in these instructions and must be operated only by **dental professionals**. Other uses than the ones stipulated will immediately invalidate the warranty.

**WARNING**

- Do follow carefully all the indications brought back in this manual.
- During all the maintenance operations disconnect the power cord from the electrical outlet.
- Do not introduce stranger objects inside the unit during the maintenance operations.
- Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks. For operation of this type, contact the technical service.
- Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

**CAUTION**

▪ **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**

- Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.
- For whichever inherent information concerning installation, maintenance and use of the unit, contact the **customer's service UGIN DENTAIRE**.

**Pictograms:**

**General warning:** failure to observe the instructions may result in damage to the equipment.

**Electrical hazard:** failure to observe the instructions may result in electric shock and serious personal injury

## 1. TECHNICAL CHARACTERISTICS

- **Size and weight:**
  - Height: 190 mm
  - Width: 175 mm
  - Depth: 190 mm
  - Weight: 4,5 Kg
- **Electric power supply:** 230 V – 50/60 Hz
- **Power:** 25W
- **Rotation speed:** 1200 revolution per minute
- **Temporization:** 0 to 30 minutes

## 2. ELECTROMAGNETIC POLISHING

A rotating magnetic field is the key to the outstanding effectiveness of **MINICLEAN 2**:

- allows in just a few minutes, the complete cleaning of dental prosthesis;
- this cleaning system does not deteriorate the prosthesis, and will slightly polish all surfaces;
- it will eliminate old cement parts as well as dental plaque, to produce a clean prosthesis;
- small in size, and easy to use, it can be integrated in a lab or dental office.

## 3. UNPACK

- Unpack **MINICLEAN 2** and check that it is in perfect order, so that any defects can be reported to the carrier.
- Remember to get the carrier to sign the delivery note

## 4. INSTALLATION AND START UP

- Place the machine in a flat surface
- After checking that the mains voltage matches that noted on the rating plate, plug the cord into a standard 10/16A socket with a ground terminal.

## 5. USE

- **Prepare the cleaning bowl:** place inside the bowl the piece to be cleaned, the needles and the water - cleaning product mixture (*for example, a washing-up liquid*)
  - **Replace the cap and cover and secure it tightly**
  - **Position the closed cleaning bowl** in the space provided for this purpose in the upper part of **MINICLEAN 2**
  - **Activate** the device through ON/OFF switch located in the back panel
  - **Select the cleaning time** using the electronic timer (0 - 30 minutes) located on the front panel (buttons + and -)
  - **Press START.** The cycle starts up after a few seconds and stops when the timer has returned to zero. To stop it before the end of the programmed time, press STOP.
- NOTE:** In the course of the cleaning cycle, the direction of rotation of the magnetic field is reserved at regular intervals, each time after a short pause.*
- **Switch off** the device through the ON / OFF switch
  - **Take the piece cleaned into cleaning bowl.** Use pliers and/or suitable gloves (*follow the cleaning product manufacturer's instructions: refer to the product data sheet and/or safety data sheet*)
  - **Rinse thoroughly in running water the pieces cleaned**
  - **Rinse the bowl and empty out the water** after each use

**ATTENTION:** after cleaning, the pieces must be rinsed thoroughly in running water. Rinse the bowl and empty out the water after each use

### 5.1. USING THE PRODUCTS

The machine is delivered with:

- 2 pack 50g of needles ECOINOX
- 1 cleaning bowl in diameter 100mm (with cap and cover)

To ensure good results, it is important to **use the proportions indicated** below:

- 1)** 50g of needles ECOINOX
- 2)** about 5mL of cleaning product (*for example, a washing-up liquid*)  
*N.B.: quantity to be adjusted according to the product used*
- 3)** water to two-thirds of the bowl height

**Before using the appliance for the first time**, we recommend washing the ECOINOX needles in the cleaning bowl containing a solution of water and about 5mL of cleaning product for 5 to 10 minutes.

**N.B.:** needles that have not been used for a prolonged period of time **may darken in colour**. In such a case, simply prepare a product + water mixture and run a cleaning cycle for a few minutes. After rinsing, the needles will look like new again.

## 6. SAFETY TIPS

Important safety tips about magnets in MINICLEAN 2:

<p><b>Warning</b></p> 	<p><b>Pacemaker</b></p> <p>Magnets could affect the functioning of pacemakers and implanted heart defibrillators. A pacemaker could switch into test mode and cause illness. A heart defibrillator may stop working. <u>If you wear these devices keep sufficient distance to magnets. Warn others who wear these devices from getting too close to magnets</u></p>
<p><b>Caution</b></p> 	<p><b>Magnetic field</b></p> <p>Magnets produce a far-reaching, strong magnetic field. They could damage laptop computer, computer hard drives, credit cards, data storage media, mechanical watches, hearing aids and speakers... <u>Keep magnets away from devices and objects that could be damaged by strong magnetic fields</u></p> <p><b>Is essential to keep watches or instruments sensitive to magnetic fields out of range of the appliance with in operation</b></p>
<p><b>Caution</b></p> 	<p><b>Nickel allergy</b></p> <p>Magnets contain nickel (coating). Some people have an allergic reaction when they come into contact with nickel. <u>Avoid perpetual skin contact with magnets. Avoid contact with magnets if you already have a nickel allergy.</u></p> <p>NOTE: during normal operation of MINICLEAN 2, no direct contact is possible with the magnets. Situation only in case of intervention for repair (change of the magnets)</p>

## 7. MAINTENANCE

	<p><b>MINICLEAN 2 must be disconnected from the main power</b> whenever maintenance work and/or cleaning is carried out.</p>
---	--

MINICLEAN 2 needs no particular servicing operation.

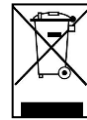
- **Clean the machine outside** with a soft, damp cloth.

*ATTENTION: do not use abrasive or solvents for cleaning outside.*

- **Rinse the bowl and empty out the water** after each use

## 8. WASTE DISPOSAL

- **Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.**
- The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used, and must be disposed of in accordance with applicable regulations.
- The black bar beneath the crossed-out bin symbol indicates that the unit was put on the market after 13/08/2005.

- **Magnets**: small amounts can be thrown out with the regular trash. Larger amounts of magnets need to be recycled as scrap metal.

*NOTE: these instructions may vary according to local regulations: refer to the legislation in force*

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