MODE D'EMPLOI - INSTRUCTIONS FOR USE

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Translation into English from original in French

D. INSTRUCTIONS FOR USE

1. SAFETY INSTRUCTIONS

1.1. USE

Steam cleaner PRESSOJET NEO with manual fill should be operated only by dental professionals.

PRESSOJET NEO must be used in the conditions and for the uses described in this manual. Any other use that will be made of it is considered inappropriate. In this case, the manufacturer declines all liability and immediately cancels the equipment warranty.

According to the product's instructions and functionalities, **it is imperative to carefully read the instructions for use before installation and use.** Retain the manual for future reference.

1.2. SAFETY INSTRUCTIONS

Pictograms' meaning used in these instructions for use:

 General warning	 Electrical hazard	 Burn hazard - Hot surface
---	---	---

Observe following safety instructions:

	- During operation and after use, the surface and steam gun may reach high temperatures: do not touch
	- The equipment must be connected to an electrical installation in accordance with the applicable standard in the country in which it is being used. This installation has to provide protection against overcurrent, overload and ground faults - Connect it directly to the mains socket. Do not use multiple sockets - Ensure that the connection cables are in perfect condition to prevent short circuits - Before any technical intervention , the equipment has to be switched off (with the breaker) and disconnected from the mains power supply - Do not spill liquid on the ventilation openings or inside the equipment
	- The equipment must be placed on a flat surface , keeping sufficient clearance around the unit to ensure proper ventilation - Do not block the ventilation openings to avoid overheating
	- Use only genuine spare parts . The use of non-original spare parts voids any warranty for your device - Do not introduce stranger objects inside the equipment during the maintenance operations. It is forbidden to modify the material without authorization - Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not included in this manual, can involve risks - For whichever information concerning installation, maintenance, and use, contact the customer's service UGIN DENTAIRE

1.3. WASTE DISPOSAL

Observe the waste disposal regulations for electrical and electronic equipment: do not dispose of with household waste.

The unit is subject to Directive 2012/19/EU on waste electrical and electronic equipment and to the laws of the country in which it is being used and must be disposed of in accordance with applicable regulations.

2. EC DECLARATION OF CONFORMITY

The equipment's EC declaration of conformity is provided as appendix of instructions for use.

Content of the declaration EC:

- Name and address of the manufacturer:
UGIN DENTAIRE. 25 rue de la Tuilerie. 38170 SEYSSINET-PARISSET – FRANCE
- UGIN DENTAIRE declares, on its own exclusive responsibility, that the product:
 - Steam cleaner with manual fill PRESSOJET NEO
 - Serial number: *see the equipment's EC declaration of conformity*
- Fulfills the essential requirements of health and safety of the following directives:
 - Machinery directive 2006/42/EC
 - Electromagnetic compatibility directive 2014/30/EU

3. TECHNICAL INFORMATION

3.1. TECHNICAL DATA

Steam cleaner PRESSOJET NEO with manual fill should be operated only by dental professionals and in the conditions described in this manual (*see chapter 1.1 and 3.2*).

TECHNICAL DATA	
Dimensions <i>(height x width x depth)</i>	270 x 240 x 360 mm (without gun)
Weight	12 kg
Voltage	220/230 V – 50/60 Hz
Power consumption	1 700 W
Stainless steel tank capacity	3 liters
Operating pressure	4.5 bars
Temperature rising time	around 20 minutes

3.2. CONDITIONS OF USE, TRANSPORT AND STORAGE

▪ **Authorised conditions of use**

- Ambient temperature: +2°C to +40°C
- Humidity range: maximum relative humidity 80%
- Ambient pressure: up to an altitude of 2000 m above sea level

▪ **Authorised conditions of storage**

- Ambient temperature: -20°C to +65°C
- Humidity range: relative humidity 10%-90%
- Ambient pressure: 500-1060 hPa

▪ **Conditions to transport**

- Wait the complete cooling before packing/transporting
- Use the original packaging only, with the protective elements
- Install the equipment in the packaging with the protective elements

4. INSTALLATION AND START-UP

4.1. UNPACK

- Unpack PRESSOJET NEO and check that it is in perfect order
- Any defects can be reported to the carrier
- Remember to get the carrier to sign the delivery note

NOTE: keep the original packaging to transport the equipment (see chapter 3.2)

4.2. INSTALLATION

- Install in a room aired. Observe the conditions indicated in chapter 3.2
- Place the equipment on a level and aired surface, easily accessible. Keep a distance from other objects around not less than 10 cm.
- Keep it away from heat sources (for example, radiators and/or other equipment that release heat). Avoid vibrations and shocks
- Do not install in a place with explosion hazard
- Its forbidden placing inflammables, toxic, volatile or explosives around the equipment

Grounding is mandatory to avoid accidents and the dangers of overheating the power line.

Due to the power of equipment: **connect it directly to the mains**, do not use multiple sockets (risk of electrical interference)

4.3. START-UP

NOTE: for marks, see chapter A – equipment description

- Remove screw cap
- Fill the tank with about a bottle and a half of distilled water (or demineralised water), in any case the filling level limits itself, do not force the bottle because it would lead to overfilling
- Replace screw cap after PRESSOJET NEO has been filled (**DO NOT FORCE ON THE CAP – Sealing joints could be damaged when too much pressed**)

IMPORTANT: never exceed the maximum level (max 3L)
To ensure long-term performance of the equipment, it is essential that only distilled (or demineralised) water be used. Do not use any other substances

- Plug in power cord to a properly grounded 230 V electrical socket
- Once the tank filled with distilled (or demineralised) water and cap closed securely, turn on power switch at the back
- Press the front panel POWER button
- Wait for the pressure to reach its working level
 - o "HEAT" light will turn from Red to Green
 - o The needle of the pressure indicator moves into the blue area

ATTENTION: Never try to unscrew the cap when the unit is switched on

- Press button on the steam handgun to release steam
- Adjust the desired steam flow with the steam adjust knob located on the steam handgun
- During the use, point the steam handgun toward job area. Do not point in another direction
- Power off the machine when you have finished using it and place the handgun

Always place the handgun in its holder after use

NOTE : Before starting the PRESSOJET NEO, it is best to check the water level inside the tank and if necessary to refill the water level. This will allow you to work with minimum lost time.

4.4. FRONT PANEL LEDS DESCRIPTION

HEAT LED – heating resistance control

- Solid **Red** when PRESSOJET NEO is heating up
- Solid **Green** when pressure is reached
- When water level is below safety, boiler LED turns OFF and heating stops

BOILER LED – Water level indicator

- **Green** when water level is above safety
- Solid **Red** for about one minute when water level is below the sensor

Both red LEDs are flashing after the minute and the buzzer sounds intermittently.

Both LEDs are steady red and the buzzer sounds intermittently when the thermal fuse trips.

In this case the machine must be sent for assistance

NOTE : PRESSOJET NEO should be used by qualified personnel only.

4.5. REFILL THE TANK

Before the water level is low, refill the tank respecting the following instructions:

1. turn off the machine
2. wait for the pressure indicator to return to zero (0)
3. carefully unscrew filler cap (to release any remaining hot steam)
Caution: hot steam – burn hazard
4. wait a few minutes longer before starting filling
5. fill the tank with around 2.5 liters of distilled water (or demineralised water), using the special container provided

WATER LEVEL: In order to avoid overheating of the device, always keep a quantity of water in the tank, ca. **2/5 of the total capacity**

HOT SURFACES : during operation, the cover and the gun steam outlet may become very hot: avoid direct contact with the skin (burn hazard)

5. MAINTENANCE

Before to carry out the maintenance work and/or cleaning:

- **disconnect from the main power supply** (*switch OFF and unplug to mains socket*)
- **wait the cooling at room temperature** after use (*burn hazard and/or damage*)

Observe the following recommendations to ensure the longevity of the device:

- When cleaning or disinfecting, make sure that no liquid gets inside
- Never carry out maintenance or cleaning operations without unplugging the power cord
- Before use, always check that the container is intact and that no electrical parts are accessible due to damage to the casing
- In the event of damage to the housing, contact the after-sales service
- The power cable must not have any cuts or damages which could lead to the risk of electric shock

- Clean the machine outside with a soft, dry cloth

- Do not use abrasive, solvents or flammables products for cleaning (outside and inside)

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